



The assisting lever is designed to fit on metal bed side bars of the **Alerta Lomond Bed** range to provide additional support when getting in and out of bed. It can also be used to reposition or raise oneself into bed. The assisting lever can be easily positioned onto the bed frame and secured by two ergonomic easy to grip hand wheels.

CLEANING & CARE

- > The assisting lever can be wiped clean with a disinfectant wipe.
- > Do not use harsh abrasives.

CAUTIONS

- > The maximum weight capacity of the grab bar is 125kg
- > The assisting lever hand wheels should be fully opened, and the clamps fitted around the side bar at a suitable position for the user
- > Care should be taken when altering the profile of the bed to prevent any risk of hand entrapment. Do not hold on the grab bar at the same time as profiling the bed
- > The hand wheels should be tightened fully so that the grab bar is clamped firmly to the side bar
- > Periodically the hand wheels should be checked and tightened as necessary