



Instruction manual



Lomond Extra High Side Rail,
Full Bed Length

ALT-LB-HSRW
Walnut

ALT-LB-HSRO
Oak



UK Version

Fitting Instructions

1) Fit the side rail onto the top of the metal bracket, resting on the crossbar with the hole in it as shown:



2) Locate the screw through the underside of the bracket and screw up into the rail and tighten:



3) Repeat this process at the other end of the side rail.



- 4) Then slide the brackets down over the existing rails on the bed.



- 5) Put the grip screws through the highest hole possible and hand tighten.



Your ALT-LB-HSRW or ALT-LB-HSRO Extra High Side Rails are now ready to use.



EU 2017/745



CS Lifesciences Europe Ltd

The Black Church
St. Mary's Place
Dublin 7
D07 P4AX
Ireland

eurep@cslifesciences.com

Rev: 2023.09

Making **cutting-edge** medical equipment **affordable**

Head Office

Alerta Medical
4 Symington Place
Riverside Business Park
Irvine
KA11 5DE
United Kingdom

www.alertamedical.com

